# PHILIPS

sense and simplicity

## Philips Respironics System One

Yves Heimans  
Klinisch Specialist Nederland  
PHILIPS Home Healthcare Solutions

**PHILIPS**  
RESPIRONICS

# Agenda:

- Software:
  - Encore Viewer 2
  - Encore Basic
- Interpretatie Encore rapport
- Sneak Preview Encore *Anywhere*

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- PC
- Encore software
- SD/smartcard lezer
- Philips Respironics CPAP toestel

# Encore Viewer:

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The screenshot shows the 'Encore Viewer' application window. The title bar reads 'Encore Viewer'. The main header features the 'RESPIRONICS' logo. Below the header, the text 'Encore Viewer™' is displayed on the left, and a 'Help' button is on the right. The main content area is titled 'Add User' and is divided into two sections: 'User Information' and 'Application Settings'. The 'User Information' section contains four text input fields: 'Name \*', 'Address', 'Phone Number', and 'Prescribing Doctor's Name'. A red line points from the 'Voer uw naam in' text box to the 'Name \*' field. The 'Application Settings' section includes a checkbox for 'Confirm user prior to therapy data transfer.' and a dropdown menu for 'Unit of Pressure' currently set to 'cm water, cmH2O'. A 'Save' button is located at the bottom right of the form area.

Steek de SD-kaart in uw kaart lees station en klik op: *Download from device*

Encore Viewer

RESPIRONICS®

Encore Viewer™

Options Help

Yves Heimans Edit User Info Add User

No usage data available. Download data from your device Download from Device

Select a Date Range for the Report

ma	di	wo	do	vr	za	zo
26	27	28	29	30	1	2
3	4	5	6	7	8	9
10	11	12	13	14	15	16
17	18	19	20	21	22	23
24	25	26	27	28	29	30
31	1	2	3	4	5	6

ma	di	wo	do	vr	za	zo
26	27	28	29	30	1	2
3	4	5	6	7	8	9
10	11	12	13	14	15	16
17	18	19	20	21	22	23
24	25	26	27	28	29	30
31	1	2	3	4	5	6

View Report

EncoreViewer - version: 2.0.23.0

©2010 Respirationics, Inc. All Rights Reserved  
www.respirationics.com

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Options Help

Yves Heimans Edit User Info Add User

No usage data available. Download data from your device Download from Device

Select a Date Range for the Report

ma	di	wo	do	vr	za	zo
26	27	28	29	30	1	2
3	4	5	6	7	8	9
10	11	12	13	14	15	16
17	18	19	20	21	22	23
24	25	26	27	28	29	30
31	1	2	3	4	5	6

ma	di	wo	do	vr	za	zo
26	27	28	29	30	1	2
3	4	5	6	7	8	9
10	11	12	13	14	15	16
17	18	19	20	21	22	23
24	25	26	27	28	29	30
31	1	2	3	4	5	6

View Report

EncoreViewer - version: 2.0.23.0

©2010 Respiration, Inc. All Rights Reserved www.respiration.com

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The screenshot shows the EncoreBasic software interface. At the top, there is a blue header with the Philips logo and 'RESPIRONICS' text. Below the header, the page title is 'Patient Detail' and there is a 'Patientenlijst' link. A green status bar indicates 'Medium gelezen - De therapiegegevens zijn opgeslagen.' Below this, there is a section for 'Naam en contactinformatie' with fields for Patient ID, First Name, Last Name, Address, Private Phone, Work Phone, Other Phone Number, and Email. There are three buttons: 'Verwijderen', 'Export', and 'Bewerken'. Below that is the 'Interaction History' section with a table:

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At the bottom right of the interface, there is a button labeled 'Van apparaat downloaden' which is highlighted by a red box and a red arrow pointing from the instruction box above.

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Below the table are three buttons: 'Verwijderen', 'Export', and 'Bewerken'. Underneath is the 'Interaction History' section with a table:

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At the bottom right of the interface is a button labeled 'Van apparaat downloaden'.

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


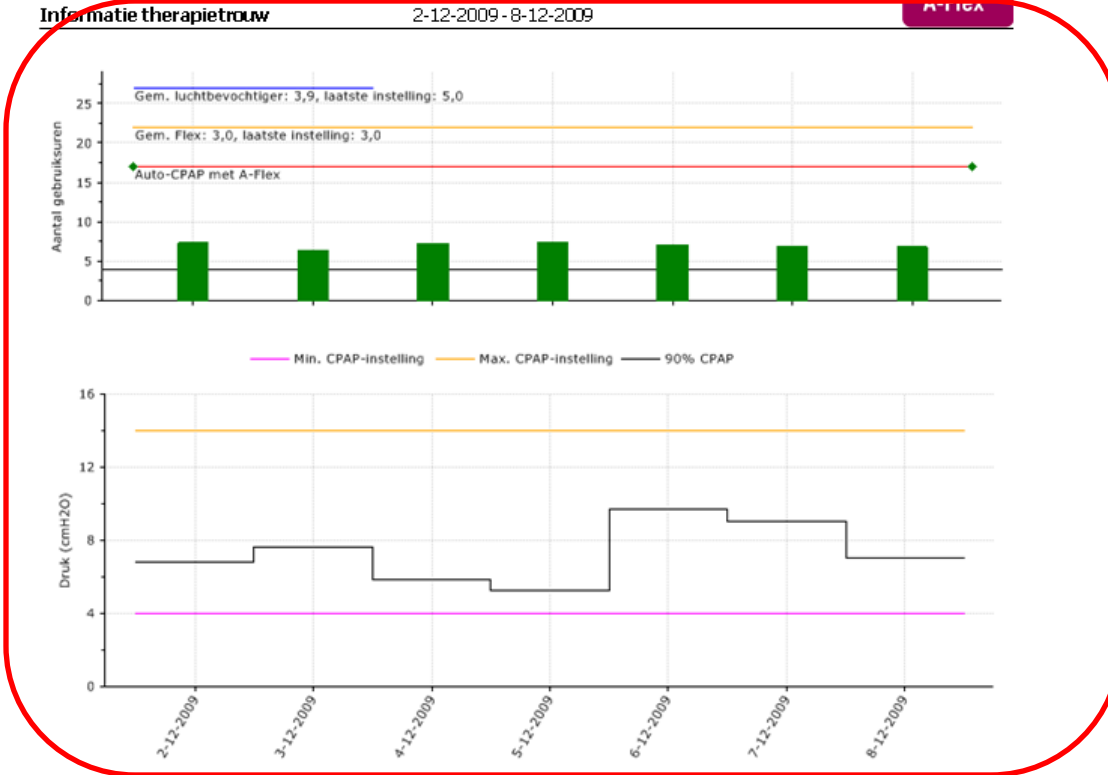
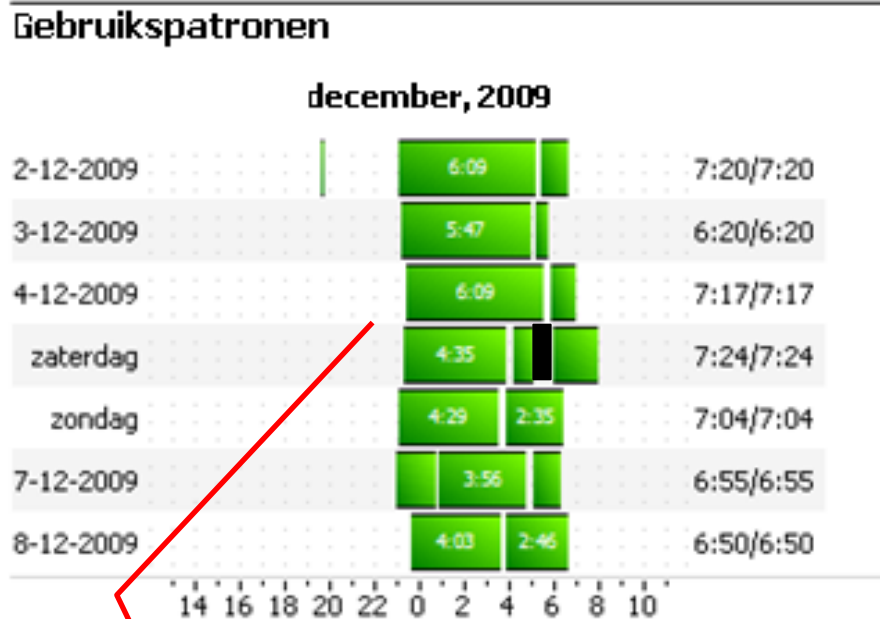
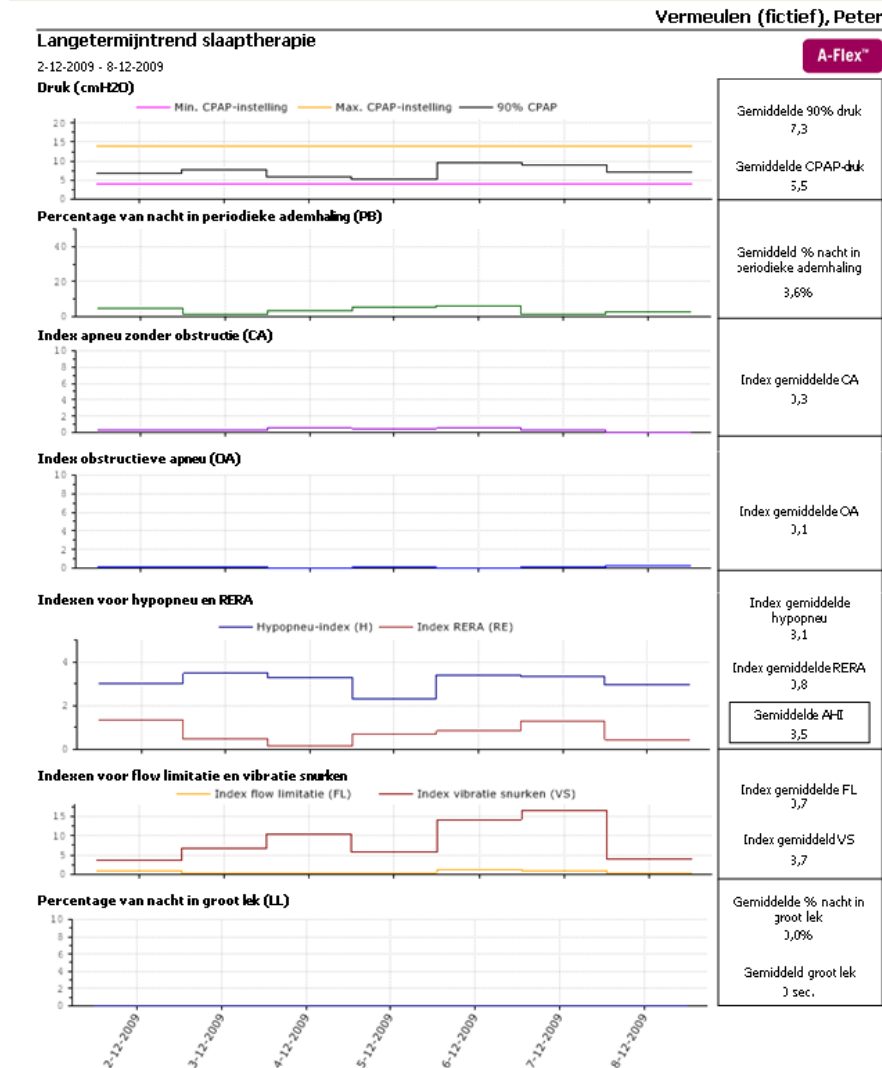
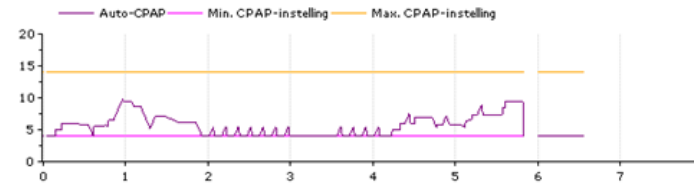
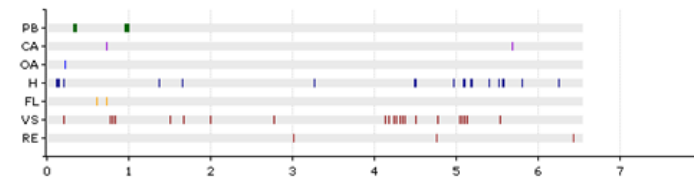
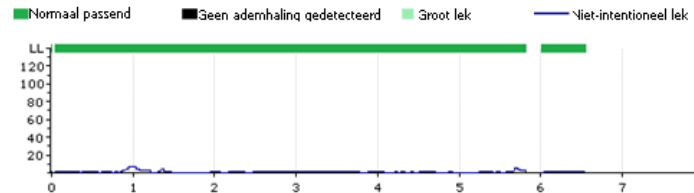
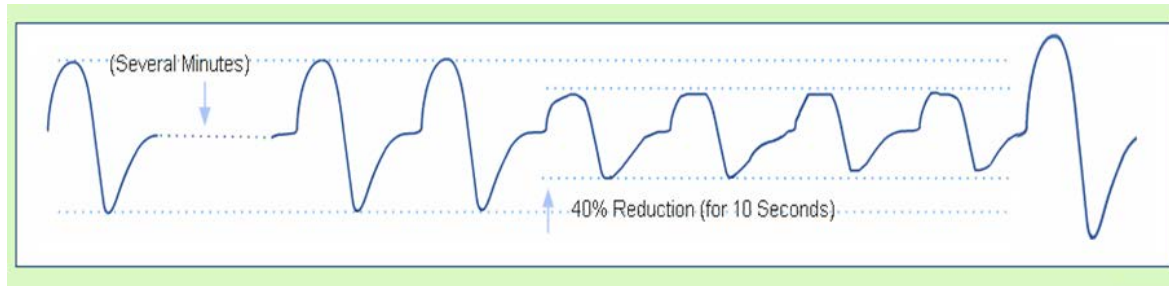
## *Apneu:*
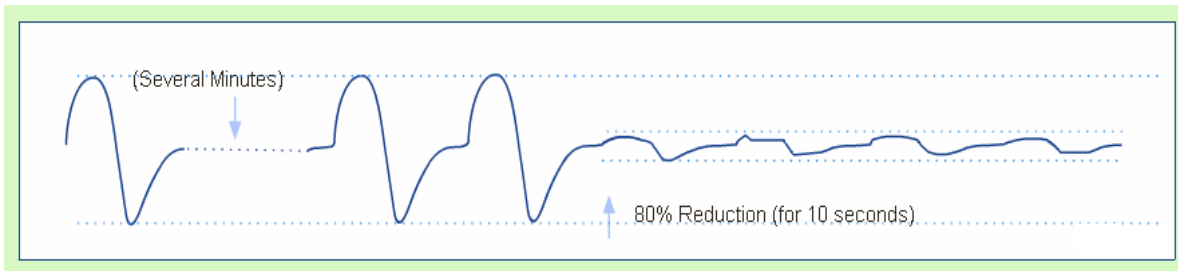
# Philips Respironics System One
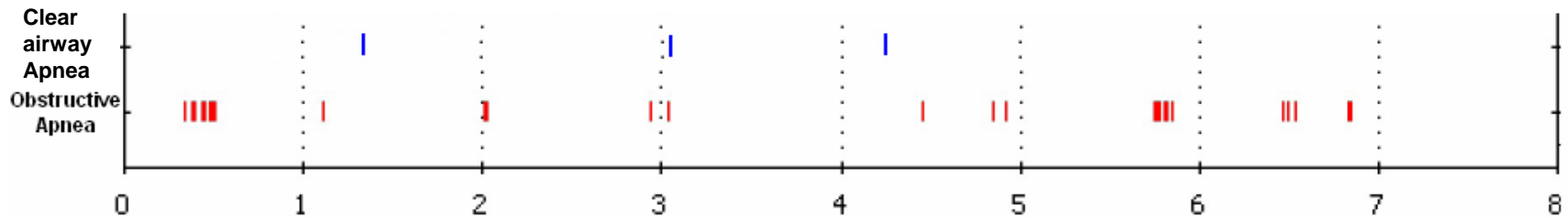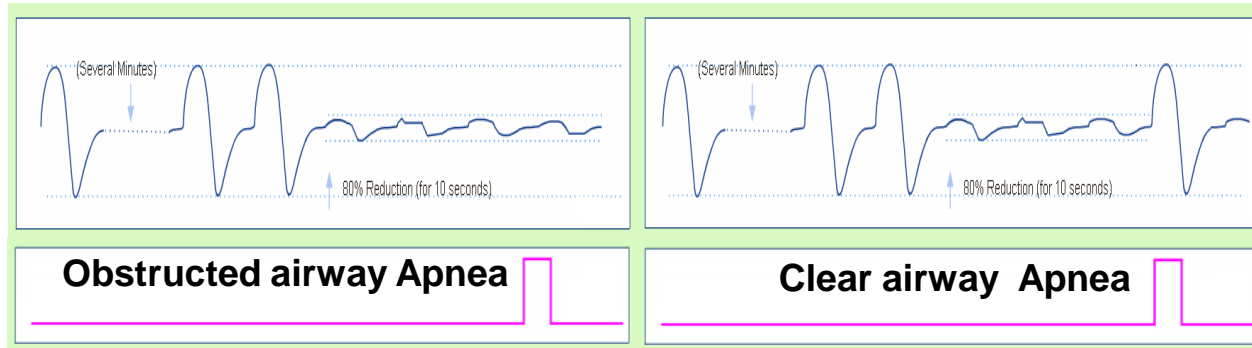
## Obstructive Apnea vs. Clear Airway Apnea

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